



Therapeutic Forceps




FUJIFILM Value from Innovation

Incision, dissection and coagulation ... ClutchCutter - your 3 in 1 ESD tool for efficient and safe therapeutic procedures

Features

- ▶ Toothed jaws to grip the mucosa membrane securely and efficient
- ▶ Rotatable distal jaws for a precise lesion approach
- ▶ Insulated outer edge for a safe procedure without damaging tissue

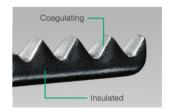

Specifications

DP2618DT ClutchCutter

Product name	Single use Diathermic Slitter DP2618DT	
Identifier	-35-	-50-
Jaw length	3.5 mm	5.0 mm
Working length	1,800 mm	
Maximum diameter of insertion portion	2.7 mm	
Forceps channel diameter of compatible endoscope	2.8 mm or more	

ESD procedures

Incision

Grip the mucous membrane by using the toothed jaws, lift up and start with the incision.



Dissection

Grip and lift up the submucosa, look to confirm, and dissect.




Coagulation

An efficient coagulation of bleeding spots is possible by using the high frequency jaws.





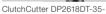

ClutchCutter DP2618DT-50-



FUJIFILM FUJIFILM Europe GmbH

Heesenstr. 31, 40549 Düsseldorf, Germany Tel.: +49 211-50 89 0, Fax: +49 211-50 89 8700 www.fujifilm.eu